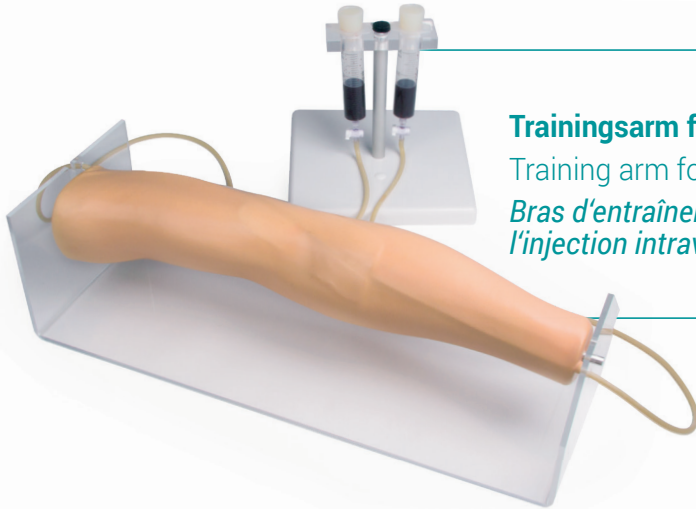
7020

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Training arm for intravenous injection

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Training arm for intravenous injection and infusion

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*Experts in medical education
since 1950*

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Congratulations

you own a high quality medical teaching product. To prevent any damage or malfunction, please notice the following instructions:

1. Removing of the protection foil from the stand (model 7020 only) The protection foil is only for transport purposes. Please remove before use.

2. How to fill and empty the tubing system

Filling

- Mount the syringe barrels in the holder and connect the vein tubes.
- Fill one syringe barrel to about $\frac{3}{4}$ with artificial blood. Make sure the shut-off valve between syringe barrels and vein are open.
- Lift up the filled syringe barrels, to allow blood to flow into the vein and the other syringe barrel using gravity. Lifting the barrel up will speed up the procedure, however, it is not necessary to fill the veins.

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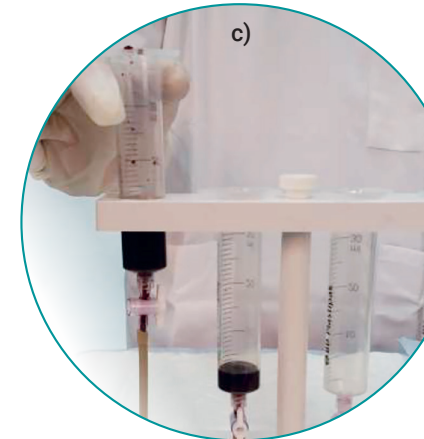
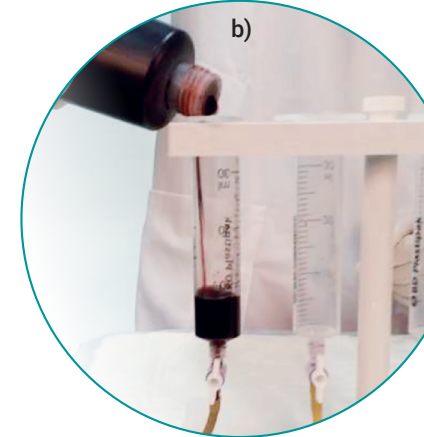
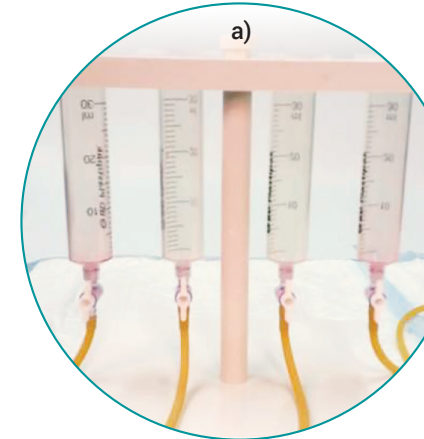
As soon as both syringe barrels are filled equally, you may start with the training. The rubber stoppers supplied shall not be used during training. They are used to close the syringe barrels while transporting the prepared arm, e.g. if you bring it to a classroom.

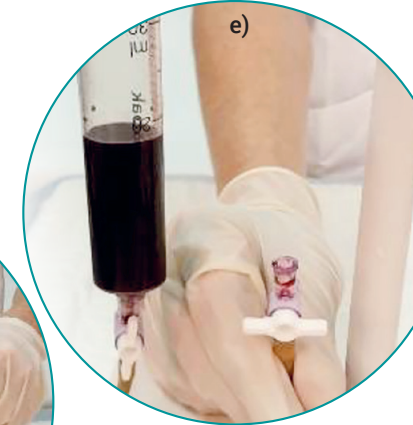
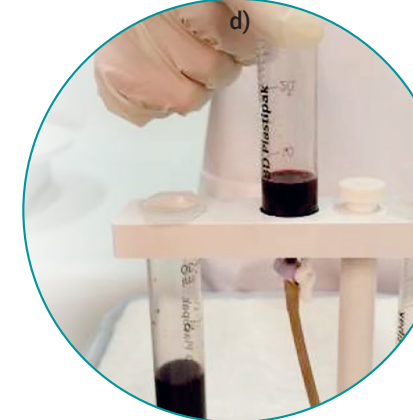
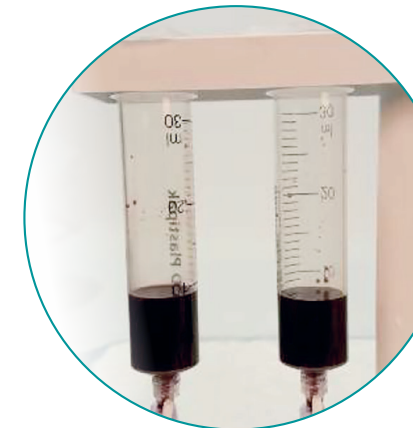
Emptying

- d) After each training the arm should always be emptied. Therefore lift one of the syringe barrels up, allowing the blood to flow through the vein into the other barrel. As soon as the syringe barrel is empty, close the shut-off valve.
- e) Remove the empty syringe barrel. Make sure the shut-off valve stays closed.
- f) Place the end of the vein in the blood bottle (the dosing insert with the small hole can be removed) and open the shut-off valve. Lift up the filled syringe barrel to allow the blood to flow through the vein into the blood bottle. As soon as the syringe barrel and most of the vein is empty, close the shut-off valve and remove the syringe barrel.

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Fill the supplied syringe with air, open the shut-off valves and press the rest of the blood out of the vein into the bottle. The vein tubing is empty. Before storage we recommend to fill the syringe with water and flush the vein system of the arm. After flushing you should use air to press the remaining water out of the tubing.

3. Changing of skin and vein

Take the arm out of the stand and remove the skin and vein. Place the new vein into the arm and pull the skin onto the arm. Use some talcum powder to make the skin slip over the arm easily.

4. Storage

Ultraviolet radiation and ozone does have bad effects on the skin and vein. Please do not store the product in sunlight or next to an ozone emerging machine like laser printer or photocopy machine.

5. Hand injection Option (only 7010)

The Model 7010 offers the possibility to inject into the hand. For this purpose the model has a separate vein tube and a short additional skin.

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Available spare parts for 7010 (arm with hand)

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English

Pos.	Art.No.	Description
1	7011	Replacement arm skin
2	7009	Replacement hand skin
3	7012	Vein tube
4	7014	Bottle with artificial blood
5	7015	Injection pads, set of 2
6	7016	Measuring tube with stopper and tap
7	7017	Arm support
8	7018/4	Holder for 4 measuring tubes
9	7019	Arm, inner core
10	7019/1	Rod for measuring tube stand
11	7019/2	Base for measuring tube stand

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English

Pos.	Art.No.	Description
1	7029	Arm, inner core
2	7023	Injection pad
3	7022	Vein tube
4	7021	Replacement arm skin
5	7024	Artificial blood
6	7027	Replacement stand for arm (plexiglass)
7	7019/1	Metal rod for stand with fixing screw
8	7019/2	Base for stand
9	7018	Clamp for 2 measuring tubes
10	7016	Measuring tube with stopper and tap

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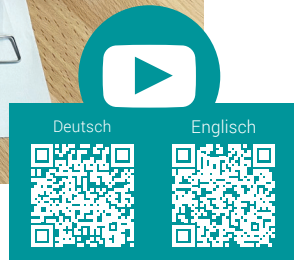
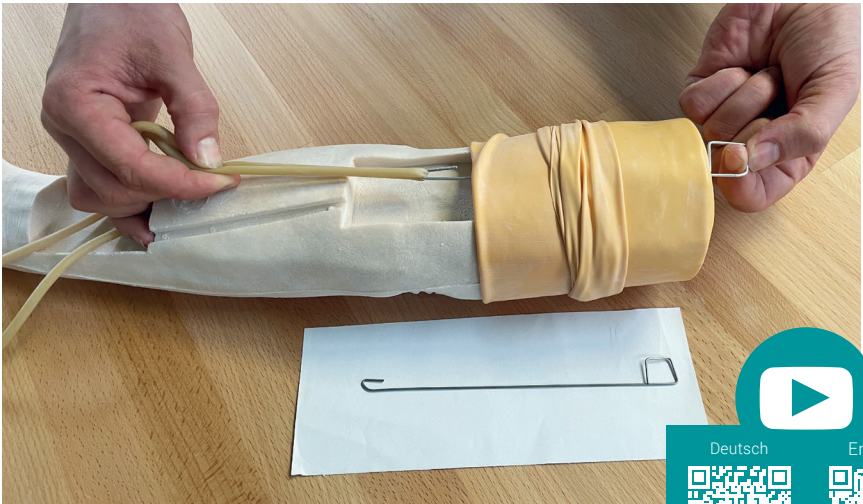
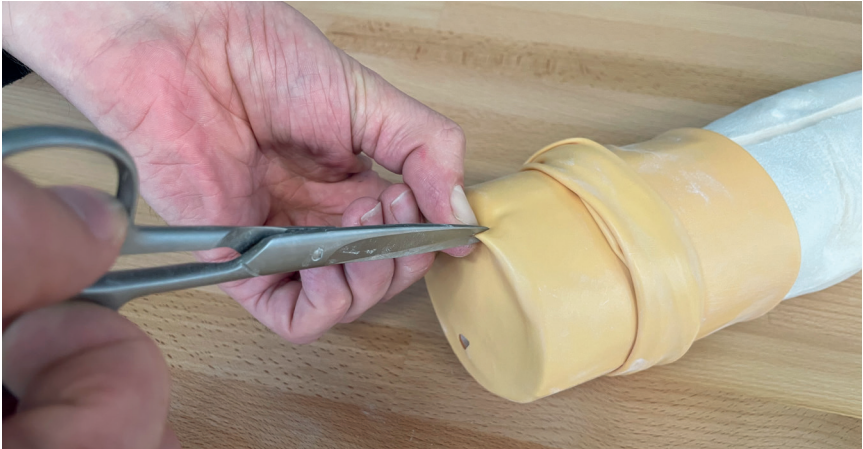
*Experts in
medical education
since 1950*

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Erlor-Zimmer GmbH & Co.KG